

Megasoft (bandage)

The Megasoft bandage lens is a therapeutic soft contact lens. The lens protects a large part of the anterior eye against effects from outside and can be used, among other things, to repair or preserve the integrity of the ocular tissue. This therapeutic soft contact lens is extra large and can be worn permanently for 4 up to 6 weeks.

Features

- Can serve as first choice in practically all indications for bandage contact lenses
- Is especially recommended for sealing the filtration bleb after trabeculectomy
- Also suitable with symblepharon
- Suitable for disorders where standard bandage lenses are regularly lost

Parameter range

Radius (r0):	10.0 mm
Power (F'v):	standard plano
Diameter (ØT):	20.5 mm

Technical information

Material:	Xylofilcon B
Water content:	76%
Centre thickness (tc):	0.38 mm
Dk	46

Care

If the lens feels unpleasant during wear a lubrication solution (Lacrimal, for example) can be applied. This solution should in any case be dropped into the eye before sleep and after waking to ensure optimal lubrication. After 4 to 6 weeks the lens must be thoroughly cleaned and disinfected.

It is advisable to use two lenses alternately for each eye, so there are no interruptions during wearing for longer periods and there is sufficient time to clean and disinfect the lens that has been worn. If the lens can no longer be cleaned and disinfected satisfactorily it must always be replaced.



Megasoft is produced by Procornea Nederland B.V.