# I-RELIEF™

## EN HOT & COLD THERAPY EYE MASK WITH THERMABEADS™

Soft, reusable, eye mask provides moist heat therapy for the relief of symptoms associated with Blepharitis, Dry Eye Disease, Meibomian Gland Disease (MGD), Styes and Chalazia.

- CONTAINS**
- 1 Eye mask 8.6 in x 3.5 in (22cm x 9 cm)
  - 1 Protective slipcover

**INDICATIONS**  
Hot & Cold Therapy Eye Mask with ThermoBeads™ for the relief of symptoms associated with:

- Blepharitis
- Dry Eye Disease
- MGD
- Styes
- Chalazia
- Headaches
- Sinus pressure
- Tension
- Swelling
- Puffiness

**DIRECTIONS**  
Use protective slipcover to keep mask clean of oils, makeup and dirt. Remove slipcover and wash as required.

**HOT THERAPY**  
Use hot therapy for relief of symptoms associated with Blepharitis, Dry Eye Disease, Meibomian Gland Disease (MGD), Styes and Chalazia.

- DIRECTIONS**
1. I-RELIEF™ mask must be at room temperature before heating.
  2. Insert I-RELIEF™ mask inside protective slipcover (optional) and place flat on clean microwave safe dish.
  3. **SET MICROWAVE POWER TO HIGH AND HEAT FOR 20 SECONDS.** Before applying, always touch the I-RELIEF™ mask with fingers first, to test for desired temperature. If the mask is too hot, allow it to cool for 1-2 minutes and test again. For additional heat, microwave in 5 second increments until desired temperature is reached. **Do not exceed a maximum of 30 seconds of heating.**
  4. Place I-RELIEF™ mask over closed eyes and secure with adjustable strap, if preferred.
  5. Wear for 3 to 5 minutes or as directed by a physician. **Do not apply for more than 20 minutes at a time.**

**COLD THERAPY**  
Use cold therapy to help relieve headaches, sinus pressure, tension, swelling and puffiness.

- DIRECTIONS**
1. Insert I-RELIEF™ mask inside protective slipcover (optional) and place in a sealable plastic bag, in the freezer for 30 minutes.
  2. Remove I-RELIEF™ mask from freezer and plastic bag, and place over closed eyes, securing with adjustable strap, if preferred. Wear for 5 to 7 minutes or as directed by a physician.

**I-RELIEF™ EYE MASK CARE**  
Remove slipcover. Hand wash only in cold water with a mild detergent. Rinse thoroughly. Gently squeeze out excess water (do not twist). Lay flat on clean, ventilated surface for a minimum of 24 hours or until fully dry. Wash immediately if exposed to food, oils, grease or other dirt.

**SLIPCOVER CARE**  
Machine wash in warm water on regular cycle with like colors, using regular detergent. Machine dry on low heat. Wash slipcover as often as necessary.

**STORAGE**  
Store I-RELIEF™ mask in cool dry place. Keep away from flames.

- WARNINGS AND PRECAUTIONS**  
Please read all information before use.
- For external use only.
  - Keep out of reach of children.
  - Do not apply for more than 20 minutes at a time.
  - Dispose of I-RELIEF™ mask if punctured or broken.
  - Always test I-RELIEF™ mask temperature with fingers prior to use on eyes.
  - Overheating or prolonged application may cause injury.
  - Do not use on infants.
  - Do not apply to sensitive skin, skin with no sensation, or open wounds.
  - Do not boil.
  - Overheating in a microwave may damage I-RELIEF™ mask.
  - Do not ingest contents.
  - Warm according to instructions. Do not apply if too hot.
  - Do not fall asleep using I-RELIEF™ mask.
  - People with diabetes, circulation disorders, skin infections, etc. should consult with their doctor before use.
  - Consult your doctor if your condition worsens or irritation develops.
  - Serious incidents should be reported to the manufacturer and local competent authority.

The I-RELIEF™ mask can be disposed of as normal waste in the garbage. The I-RELIEF™ box and insert can be disposed of in paper recycling bins.

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I-RELIEF™ is a trademark of I-MED Pharma Inc. ©2025 I-MED Pharma Inc.
I-RELIEF™ est znakiem towarowym firmy I-MED Pharma Inc ©2025 I-MED Pharma Inc.
**ARAZY GROUP GmbH**
**Grafton Optical Company Ltd**